



# Care Instructions for Enova Loupes

How to care for your loupes



Correct: Always hold your loupes by the frame.



Incorrect: Never hold your loupes by the oculars.



Hold under running water for a preliminary rinse.



Apply a drop of mild soap and rub into glasses.



Rinse under running water.



Carefully dry with a lint-free cloth or soft paper towel.



Correct: Always rub the ocular inwards (toward the nosepiece).



Incorrect: Never rub the ocular outwards (toward the arms).



Apply a small amount of disinfectant with a lint-free cloth or soft paper towel.



Incorrect: Never spray disinfectant straight onto glasses.



Only use non-alcohol disinfecting agents on your loupes.

For further information on the cleaning, care, and handling of Enova surgical and dental loupes and protective glasses, please see reverse side.

World Headquarters  
Enova Illumination  
6305 Sandburg Road, STE 100  
Minneapolis, MN 55427

© 2022 Enova Illumination, Inc. All trademarks are the property of Enova Illumination, Inc. unless otherwise noted.





# Care Instructions for Enova Loupes

Dear Customer,

We are delighted that you have chosen Enova for your custom loupes. The frame and lens of your loupes require careful cleaning to prolong its useful life.

Please Note: After use, small particles tend to remain on the surface of the frame and carrier lens. If you rub it with a cloth or your fingers, these dirt particles can cause irreparable damage.

## CLEANING

- 1** When handling, always hold your loupes by the frame, making sure to avoid applying firm pressure to the oculars.
- 2** Run glasses under lukewarm water to remove any hard particles.
- 3** Once free of debris, use household dishwashing liquid to thoroughly clean the entire product.
- 4** After rinsing, dry the glasses with the special cleaning cloth provided with your product, a lint-free cloth, or a soft paper towel.

## DISINFECTING

- 1** Disinfect your loupes after thorough cleaning and drying by using alcohol free\* disinfectants.
- 2** Spray disinfecting agent onto a lint-free cloth and use it to clean the lens and oculars.
- 3** Never spray disinfectant directly on your loupes to prevent the agent from settling into spaces between components of your loupes.

\*Check the disinfecting agent ingredients by looking at the product packaging or bottle before

## CHARGING

- 1** Batteries for loupe lights can be charged over night, or up to 48 hours without causing damage to the power pack.
- 2** We recommend not leaving batteries connected to charger long-term to avoid depleting battery performance.
- 3** DO NOT charge batteries in a surgical suite.