


BROKEN INSTRUMENT REMOVAL KIT

zumax

Broken Instrument Removal Kit



Extractor Tip Options

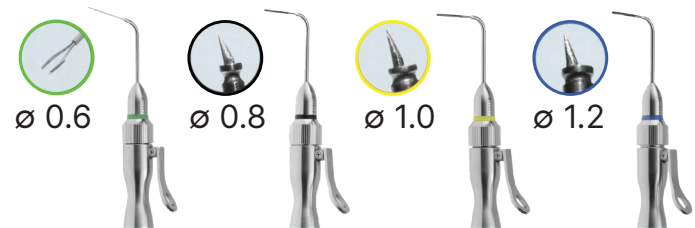

Advantages:

- Easy to use
- Limits the amount of dentin removed from the root canal wall.
- Using microscopic tweezers for instrument removal reduces the risk of ultrasonic tip fracture
- Microscopic tweezers are preferable for instrument retrieval in the coronal third

Contents:

- 3 Micro-Trephine Burs
- 1 Extractor Handle
- 3 Extractors
- 1 Foreign Forceps
- 1 Autoclavable Tray




MICRO DENTAL TWEEZER

zumax

Micro Dental Tweezer

Instructions:

1. When any foreign matter is discovered during a procedure, insert tweezer tip into the canal to remove it
2. After the foreign matter is removed, use the cleaning adapter to clean the interior of the tweezer
3. Autoclave the entire tweezer and tip after each use

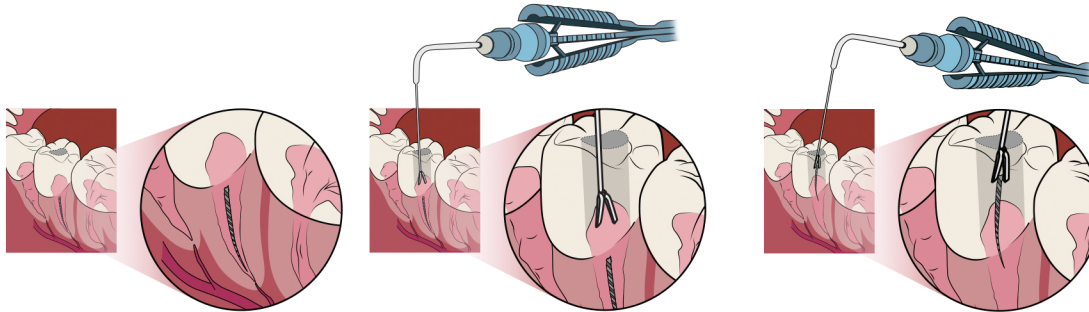

Advantages:

- 360 degree adjustment
- Accurately clamps foreign material with firm grip, high stability, and reliability

Contents:

- 1 Tweezer Head
- 1 Handle
- 1 Cleaning Adapter

Specifications: 0.9mmx20mm



Micro-Claw
(0.9mm x 20mm)

To learn more about how Enova can help you see better, so you can perform better, visit www.goenova.com, call 651.236.8857, or email sales@goenova.com

F20414C
© 2025 Enova Illumination, Inc. All trademarks are the property of Enova Illumination, Inc. unless otherwise noted.
Zumax® is a registered trademark of Zumax Medical Co., LTD

Visit our online store at store.goenova.com

World Headquarters

Enova Illumination
6305 Sandburg Road, STE 100
Minneapolis, MN 55427

ISO 13485: 2016 Certified QMS
MDR Class 1 Certified